

The DizzyFIX device is safe, simple to use, reliable and effective.

For more information about this or other products from Clearwater Clinical Limited please contact us:



Research:

Three clinical trials have been conducted at the University of Western Ontario regarding the DizzyFIX device. The concept of a head-worn visual feedback device has been proven with healthy volunteers in a randomized clinical trial. In addition, the reliability of the device in predicting a correct maneuver has been demonstrated. Finally, 88% of patients who used the DizzyFIX for the self treatment of BPPV were successful. This cure rate did not differ significantly from the 83% success rate in patients who had treatment maneuvers guided by an ENT physician. These three studies have found the DizzyFIX device to be safe, simple to use, reliable and effective.

More information for doctors is available at www.dizzyfix.com/physician



+ Clearwater Clinical Limited™
100 - 111 5th Ave SW,
Suite 258
Calgary AB, T2P 3Y6
CANADA
1-877-349-9934

Vendor Contact Information

Copyright 2005-2009 Clearwater Clinical Limited™



www.dizzyfix.com

DFPTB-01-20070601-01 PDF



Anyone can treat dizziness*
with the DizzyFIX



K081602



Effective Treatment for Dizziness - Anywhere



Everyone has experienced dizziness. The most common cause of dizziness related to the ear is Benign Paroxysmal Positional Vertigo (BPPV). BPPV is caused by free-floating crystals in the inner ear. These crystals cause spinning dizziness when disturbed by head motion.

There is, however, an effective treatment for BPPV. The DizzyFIX enables anyone to perform a "Particle Repositioning Maneuver". This maneuver cures most cases of BPPV.

Whether being used by a patient at home, a doctor or other health professional, this device helps diagnose and treat BPPV.

+ Dizziness can be debilitating.

Benign paroxysmal positional vertigo (BPPV) is the most frequent cause of peripheral vertigo. The incidence of BPPV increases with age and has been reported as greater than 9% in people over 60 years old. Recurrence rates have been reported in up to 50% of patients.

Do you have the symptoms of BPPV?

- I am dizzy when I roll over or move my head in certain ways
- The sensation I experience lasts only for a few minutes, but nausea can last longer
- The room spins when I lie down

Complete Instructional DVD



FDA Cleared

The DizzyFIX Includes:

- DizzyFIX Device
- DizzyFIX Hat
- Written Instructions
- Instructional DVD
- Quick Reference Card

